C.LENS[®] plus

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INDICATIONS

C.LENS Plus[®] multifunctional solution lens care moisturizes, cleans, disinfects, removes protein, rinses, lubricates and preserves your contact lenses every day.

USAGE TIPS

The maintenance of your contact lenses in four steps:

1 / Cleaning: Remove the right lens, place it in the palm of your hand, pour about 3 ml C.LENS Plus[®] solution by covering the lens and massage gently for about 20 seconds.

2 / Rinsing: Gently rinse the lens with C.LENS Plus[®] solution taking care to remove any visible deposits and place the lens in the right compartment of the case. Do the same for the left lens by cleaning, rinsing and placing it in the left case compartment.

3 / Disinfection and storage: Fill the lens case with C.LENS Plus[®] solution. Shake lightly to spread the C.LENS Plus[®] solution over the entire surface of the lenses and soak for at least 4 hours to thoroughly disinfect your lenses, which will then be ready for re-use.

4 / Rinsing and lubrication before placing the lenses: Before putting your lenses, please rinse them with C.LENS Plus[®] solution in order to eliminate any residual deposits. Then pour a few drops of C.LENS Plus[®] solution onto each lens just before laying.

- Respecting these steps is very important to preserve your lenses and your eyes.
- Before handling your lenses, be sure to wash and dry your hands thoroughly.
- It is important to empty the used solution from the case after each maintenance cycle and always use fresh C.LENS Plus[®] solution.

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Opalia Pharma S.A., Recordati Group