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Crème dermique en flacon airless de 70 g.

Composition :

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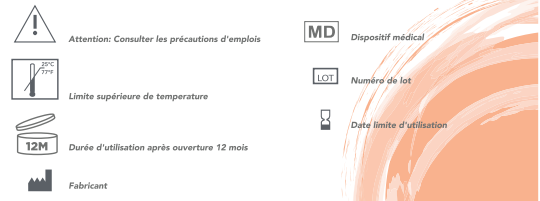
Dermal cream in a 70 g airless bottle.

Composition:

Hyaluronic Acid, sodium salt (0.2%), Purified Water, PEG-400 Stearate, Decyl Oleate, PEG-8 Beeswax, Emulsifying Wax, Glycerin, Sorbitol, Sodium Acetate Trihydrate, Perfume, Methyl Parahydroxybenzoate, Sorbic Acid, Propyl Parahydroxybenzoate, Sodium Hydroxide.

Properties:

Hyaluronic acid is a mucopolysaccharide of biological origin, which is an essential component responsible for the hydration, reparation and regeneration of the dermis.

IALUSKIN® Cream is a medical device. It involves its main constituent, hyaluronic acid, to act in all stages of the wound healing process and regeneration.

By stimulating the formation and reparation of dermal tissues, IALUSKIN® Cream accelerates healing and re-epithelialization mechanisms of minor lesions and small size burns.

Indeed, IALUSKIN® Cream has moisturizing properties that allow it to prevent skin drying, keeping the treated area in a moist environment, promoting tissue regeneration and soothing itchy irritated skin.

Indications:

IALUSKIN® is a cream with triple action «reparative - moisturizing - soothing» intended to help the healing of damaged and irritated skin.

It is indicated in the following cases:

- Grazes, minor cuts, pimple traces, skin irritations, chapping.
- Moisturizing of damaged skin, particularly those attacked by the sun.
- After small dermatological interventions for aesthetic purposes: peeling, laser, injection...

Instructions for use and Dosage:

IALUSKIN® Cream should be applied 2 to 3 times a day, in a thin layer of approximately 2-3 mm, uniformly on the whole area to be treated.

It is recommended to wash your hands before and after each use of IALUSKIN® Cream.

Before application, it is necessary to clean and disinfect the irritated skin or lesions and their contours.

It is especially recommended to remove residual cream, left on the skin from a previous application.

It is advisable to continue the applications until the wound is completely healed.

Do not exceed 30 days of treatment.

On the first use, it is often necessary to press the pump many times, repeatedly, until the cream comes out of the diffuser.

In the case of a person who is not self-sufficient, recourse to the assistance of another person is possible.

Side and troublesome effects:

In very rare cases, local eczema type hypersensitivity reactions have been observed.

Contraindications:

- Do not use IALUSKIN® Cream in case of known hypersensitivity to any of its constituents.
- If local hypersensitivity reactions persist, consult your doctor.
- Do not use IALUSKIN® Cream in the case of serious, deep and very soiled wounds.
- Avoid all contact with the eyes and mucous membranes.
- Ask your doctor or pharmacist for advice during pregnancy or while breastfeeding, before using IALUSKIN®.

Precautions for use / Warnings

- Keep out of reach of children.
- For external use only. Do not swallow.
- Since interactions with other products are not known, it is recommended to seek medical advice before

using IALUSKIN® Cream in combination with other medical devices or local-use medications.

- Follow precautions and methods for cleaning and disinfecting wounds and their contours.
 - Before each use remove the glued cream on the diffuser end, and clean its tip carefully.
 - Do not apply the tip of the diffuser of the Airless bottle directly on the wound or treatment area.
 - Close the product immediately after use.
 - The expiry date relates to a closed and stored under proper conditions.
 - Do not dispose of the medical device in the sewage system or with household waste. Ask your pharmacist what to do with medical devices that are no longer used.
- These measures will help protect the environment.

IALUSKIN® is a medical device. Not for cosmetic use.

Storage conditions:

Do not exceed the expiry date stated on the outer packaging.

This medical device should not be stored above 25°C.

To be used within 12 months after opening.

Leaflet revision date : February 2023

Caution : Consult the precautions of use.

Upper temperature limit

Shelf life after opening 12 months

Manufacturer

Medical device

Batch number

Expiry date

Laboratoires Opalia Pharma S.A., Recordati group
Z.I. Kalaât El Andalous 2022 Ariana - Tunisia - Tunisia