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Please read this entire instructions for use's leaflet carefully; it contains important information for the user. Keep this leaflet; you may need to read it again.

ACC0007/11

Form and presentation

IALU Proct® is in the form of suppositories for rectal use. The box contains 10 suppositories of 2 g.

Composition

Hyaluronic acid sodium salt (5mg), hemisynthetic glycerides, Centela Asiatica, Calendula officinalis, Aloe vera, Melaleuca Essential oil, BHT s.q.f. 1 suppository.

Properties

Hyaluronic acid is a mucopolysaccharide of biological origin, which is an essential component of many body tissues responsible for their hydration, repair and regeneration. Several studies have demonstrated the ability of hyaluronic acid to accelerate and improve the process of epithelialization and tissue repair.

IALU Proct® suppository is a medical device. It involves its main constituent, hyaluronic acid, which promotes healing by accelerating the process of repairing anal lesions.

Indications


IALU Proct® Suppository is an adjuvant treatment of anal lesions repair processes following:

- Proctologic surgery
- Internal or external hemorrhoids
- Proctitis
- Cryptitis (inflammation of the rectum and / or anus or anal crypts)
- Anal rhagades
- Cracks
- Perianal fistulas.

Mode and duration of use

Remove the suppository from its package by pulling the two pre-cut tabs of the blister. Insert the suppository into the rectum.

For hygiene reasons, it is recommended to wash your hands before and after each use of this medical device.

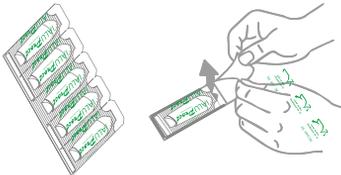Use a suppository a day, preferably in the evening before going to bed or as prescribed by the doctor.

The duration of use depends on the evolution of symptoms; if necessary, IALU Proct® suppositories can be used for prolonged periods.

Contraindications

- Do not use IALU Proct® in case of individual hypersensitivity to any of the product's constituents.

Interactions

Do not use in combination with other rectal medical devices or medicines without first consulting a doctor.

Precautions for use / Warnings

- Prolonged use of any product for local use may cause sensitization. In this case, it is advisable to discontinue use of the treatment and consult a doctor to initiate appropriate treatment. If any other side effects occur, please inform your doctor or pharmacist.

- Keep out of reach of children.
- If you are pregnant or breastfeeding, ask your doctor or pharmacist for advice before using IALU Proct® suppository.
- Do not swallow
- Avoid any contact with eyes.
- The expiry date refers to a product in its closed packaging and stored under appropriate conditions. Do not use the product after the expiry date or if it is open or damaged.
- Do not dispose of the medical device in the sewage system or with household waste. Ask your pharmacist what to do with medical devices that are no longer used. These measures will help protect the environment.

Storage conditions

Keep the medical device away from light and heat, preferably at a temperature between 4°C and 25°C.

Text's revision date : May 2025.

Manufacture: Laboratories Opalia Pharma S.A., Recordati group
Z.I. Koladji Et Anadou 2022 Anania - Tunisie - Tunisia