Please read carefully the entire instructions of this leaflet: it contains important information for the user.
Keep this leaflet; you may need to read it again.

Stick pack
Medical device recommended in case of gastroesophageal reflux.

Form and presentation :
IALUSopH® comes as an oral gel in a 10mL stick pack, box of 20.

Composition :
Hyaluronic acid sodium salt, sodium chondroitin sulfate, polyvinylpyrrolidone, xylitol, sodium benzoate, potassium sorbate, cherry flavor, water and poloxamer 407.

Properties :
Medical device recommended in case of gastroesophageal reflux.
IALUSopH® is a medical device containing a combination of hyaluronic acid and chondroitin sulfate which together provide a mechanism of action that protects and helps to repair the gastro-oesophageal mucosa. Thanks to its highly bioadhesive component (poloxamer 407), IALUSopH® allows a good adhesion of hyaluronic acid and chondroitin sulfate on the surface of the oesophageal wall, thus contributing to the protective effect against the action of the hydrochloric acid and pepsin produced by the stomach.
Hyaluronic acid coats the esophageal mucosa limiting the contact of refluxed acid and pepsin with the epithelial surface.
Chondroitin sulphate fulfills important biological functions such as; cell regeneration, inflammation reduction & wound repair and promotes therefore the repair process of the gastro-oesophageal mucosa.
The Poloxamer 407 formulation is a

bioadhesive that prolongs the contact time of chondroitin sulphate and hyaluronic acid with the esophageal mucosa.

IALUSopH® contains xylitol, a non-cariogenic natural sweetener and does not induce significant changes in blood sugar levels. The product is also suitable for diabetics. It has no drug interaction.

Indications :
IALUSopH® is a medical device that exerts a mechanical action promoting rapid improvement in symptoms associated with gastroesophageal reflux : heartburn, epigastric pain, regurgitation, irritating cough and dysphonia.

Mode and duration of use :
Take the contents of one stick pack after the main meals and at bedtime, or as recommended by the doctor.

Contraindications :
There are no contraindications, except in cases of hypersensitivity to any of the components. In this case, stop treatment and contact your doctor. Although there are no known side effects, this product is not to be used during pregnancy or in the first months of breastfeeding. It is always advisable to check with your doctor before use.

Precautions of use/ Warnings:


- Do not exceed the recommended dose.
- Contact your doctor if symptoms persist.
- Due to the lack of clinical data in children under 12 years of age, do not use this product in children under 12 years of age.
- Keep out of the reach of children.
- The expiry date refers to a product in its closed packaging and stored under appropriate conditions. Do not use the product after the expiry date or if it is open or damaged.
- Do not freeze.

- Do not dispose of the medical device in the sewage system or with household waste. Ask your pharmacist what to do with medical devices that are no longer used. These measures will help protect the environment.

Conservation :
Keep the medical device tightly closed away from light and heat, preferably at a temperature not exceeding 25°C.

Leaflet revision date:
March/2025.

Manufactured by OPALIA Pharma
RECORDATI Group :

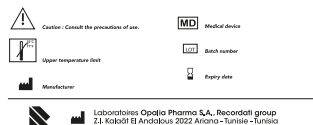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