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-Please read carefully the entire instructions of this leaflet: it contains important information for the user.

-Keep this leaflet; you may need to read it again.

AC1615/V3

SILISKIN®

Gel de silicone / Silicon gel

Form and presentation:

SILISKIN® silicon gel is presented out of 15 g airless bottle.

Composition:

Polysiloxanes, silicon dioxide.

Properties :

SILISKIN® is a topical transparent silicon gel that dries quickly, helps maintain the skin hydrated and normalizes excessive collagen formation that contributes to abnormal scarring.

SILISKIN® forms a bond with the stratum corneum forming a protective barrier against chemical, physical and microbial invasion of the scar.

SILISKIN® creates an invisible shield that feels like a second skin and helps reduce the appearance of scars. It can reduce redness and discolouration, soften and flatten new and old hypertrophic and keloid scars resulting from surgical procedures, injuries and burns.

SILISKIN® can be used on joints, flexures, hairy areas and large area scars such as those caused from burns or major trauma. SILISKIN® can be used in conjunction with pressure garments or other scar therapy options.

Indications:

SILISKIN® is indicated for the treatment of old and new scars, including hypertrophic scars and keloids, resulting from surgical procedures, trauma, wounds and burns. SILISKIN® is suitable for use on children and for people with sensitive skin.

Mode and duration of use:

- Before any application, clean and dry the skin area to be treated.

- On the first use, it is often necessary to press the pump many times, repeatedly, until the gel comes out of the diffuser.

- Apply SILISKIN® on the scar in a very thin layer in one step: do not rub and allow to dry for 4 to 5 minutes.

- Apply SILISKIN® twice a day (morning and evening).

- For maximum effect, SILISKIN® should remain in contact with the skin for 24 hours.

- Once dry, SILISKIN® can be covered with pressure garments, sunscreen or cosmetics.

If SILISKIN® has not dried in 4 or 5 minutes, you've probably used too much. Gently remove the excess to allow drying and to avoid staining your clothes.

The minimum duration of treatment is 2 to 3 months. Larger and older scars may take longer and continued use is recommended if improvement is still seen after the first 3 months.

Tips for drying:

SILISKIN®'s drying process is assisted by the temperature difference between the gel and the skin. To reduce drying time in hotter climates, keep SILISKIN® in the refrigerator. In colder weather, use the low setting on a hair dryer to reduce the drying time.

Contraindications :

- Do not use SILISKIN® in case of known hypersensitivity to any of its constituents. If local hypersensitivity reactions persist, consult your doctor.

- Do not apply SILISKIN® on open wounds and third degree burns.

- Do not apply SILISKIN® on other skin treatments without the advice of your doctor.

- Avoid all contact with eyes and mucous membranes.

Precautions of use/ Warnings:

- Keep out of reach of children.

- SILISKIN® should not be used on dermatological conditions that disrupt the integrity of the skin.

- SILISKIN® may stain clothing if it is not completely dry.

- For external use. If irritation occurs, discontinue use and consult a doctor.

- Do not use the product after the expiry date (EXP) on the outside package or if it is opened or damaged.

- Do not dispose of the medical device in the sewage system or with household waste. Ask your pharmacist what to do with medical devices that are no longer used. These measures will help protect the environment.

Storage conditions:

Keep the bottle away from heat, preferably at a temperature not exceeding 25 ° C.

Leaflet revision date:

October 2023.

Caution - Consult the precautions of use.

Upper temperature limit

Manufacturer

Medical device

Lot batch number

Expiry date

Laboratoires Opalia Pharma S.A., Recordati group
ZI Kadadi El Andalous 2022 Arzana - Tunisie - Tunisia